

Classical Swine fever virus antigen rapid test card

**INSTRUCTION MANUAL
FOR PRODUCT No: LT01002AYSL**



Arsh Biotech Pvt. Ltd.

308, Aggarwal City Mall, Road No.44,
Pitampura, Delhi-110034, India
Toll Free: 1800-2745-745
Mobile: +91-98105-21400 | Fax: +91-11-42208444
info@arshbiotech.com

MANUAL VERSION 1.01

PRINCIPLE

This rapid test is an immuno-chromatographic test, using colloidal gold immunoassay method, to detect antibody against swine fever virus in whole blood, serum, and plasma. The whole test only need 20 mins. Easy to operate, fast, accurate, high sensitivity, intuitive result, easy to judge.


MATERIALS SUPPLIED WITH THIS KIT

1	CSFV Ag test card	50 pieces
2	Disposable gloves	1 pair
3	Manual	1 piece

TEST PROCEDURE

- 1) Place the test card on the desktop flatly, mark it.
- 2) Add 2 drops (70-100 μ l) serum/ plasma/whole blood sample into the well on the test card.
- 3) Be static at room temperature, then judge the result in 20 minutes.

RESULTS JUDGMENT

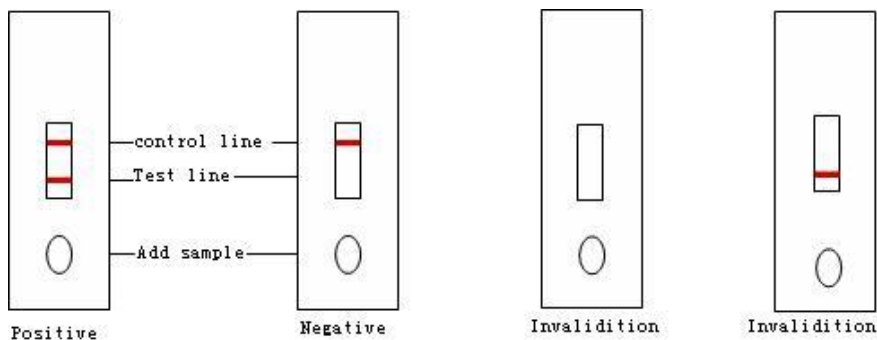

1. Positive: control line and test line are both seen wine red.
2. Negative: only control line is seen wine red.
3. Invalidation: control line isn't seen wine red.

NOTICE

Use the test card as soon as possible once open the aluminum foil, or the test card will be invalidated, store at normal temperature and the shelf life is 18 months.

If use whole blood as sample, the sample moving speed may be slow, add 1 drop PBS solution 1 minute later after adding sample to help sample moving.

STORAGE AND EXPIRY DATE

Store at 2 - 30°C and the shelf life is 24 months.