

**Leukemia Gene JAK2 V617F Mutation Real Time RT-PCR Kit**

User Manual

For In Vitro Diagnostic Use Only

REF LT017920RT

For use with LightCycler1.0/2.0Instrument



Life Technologies (India) Pvt. Ltd.  
306, Aggarwal City mall, Opp. M2K Pitampura, Delhi110034  
(India). Ph: +91114220800,  
42208111, 42208222, Mob: +919810521400,  
fax: +911142208444  
email: customerservice@lifetechindia.com Website: www.lifetechindia.com

**1. Intended Use**

Leukemia Gene JAK2 V617F Mutation Real Time RT-PCR Kit is used for the detection of Leukemia Gene JAK2 V617F Mutation in leukocyte by using real time PCR systems.

**2. Principle of Real-Time PCR**

The principle of the real-time detection is based on the fluorogenic 5' nuclease assay. During the PCR reaction, the DNA polymerase cleaves the probe at the 5' end and separates the reporter dye from the quencher dye only when the probe hybridizes to the target DNA. This cleavage results in the fluorescent signal generated by the cleaved reporter dye, which is monitored real-time by the PCR detection system. The PCR cycle at which an increase in the fluorescence signal is detected initially is proportional to the amount of the specific PCR product. Monitoring the fluorescence intensities in real-time allows the detection of the accumulating product without having to re-open the reaction tube after the amplification.

**3. Product Description**

The JAK2 gene was first cloned in 1989 and is a member of a family of four Janus kinases 1, 2 and 3 and tyrosine kinase 2. JAK2 V617F is found in the majority of patients with Ph- myeloproliferative neoplasms (MPNs) and chronic neutrophilic leukemia (CNL). However, it has also been found in many other hematological diseases.

Leukemia Gene JAK2 V617F Mutation Real Time RT-PCR Kit contains a specific ready-to-use system for the detection of the leukemia gene JAK2 V617F mutation using RT-PCR (Reverse Transcription Polymerase Chain Reaction) in the real-time PCR system. The reaction is done in one step real time RT-PCR. The first step is a reverse transcription (RT), during which the mutant gene is transcribed into cDNA. Afterwards, a thermostable DNA polymerase is used to amplify the specific gene fragments by means of PCR (polymerase chain reaction). Fluorescence is emitted and measured by the real time systems optical unit during the PCR. The detection of amplified fragment is performed in fluorimeter channel 530nm with the fluorescent quencher BHQ1. An external positive control (1x10<sup>7</sup> copies/ml) supplied, allows the determination of the gene load. For further information, please refer to section 9.2 Quantitation.

**4. Kit Contents**

Rfd.	Type of reagent	Presentation 25rxns
1	JAK2-V617F Super Mix	1 vial, 380µl
2	RT-PCR Enzyme Mix	1 vial, 28µl
3	Molecular Grade Water	1 vial, 400µl
4	JAK2-V617F Positive Control (1x10 <sup>7</sup> copies/ml)	1 vial, 30µl

Analysis sensitivity: 1x10<sup>3</sup> copies/ml; LOQ: 2x10<sup>3</sup>~1x10<sup>8</sup> copies/ml

Note: Analysis sensitivity depends on the sample volume, elution volume, nucleic acid extraction methods and other factors. If you use the RNA extraction kits recommended, the analysis sensitivity is the same as it declares. However, when the sample volume is dozens or even hundreds of times greater than elution volume by some concentrating method, it can be much higher.

**5. Storage**

- All reagents should be stored at -20°C. Storage at +4°C is not recommended.
- All reagents can be used until the expiration date indicated on the kit label.
- Repeated thawing and freezing (> 3x) should be avoided, as this may reduce the sensitivity of the assay.
- Cool all reagents during the working steps.
- Super Mix should be stored in the dark.

**6. Additionally Required Materials and Devices**

- Biological cabinet
- Real time PCR system
- Desktop microcentrifuge for "appendorf" type tubes (RCF max. 16,000 x g)
- Vortex mixer
- RNA extraction kit
- Real time PCR reaction tubes/plates
- Cryo-container
- Pipets (0.5 µl – 1000 µl)
- Sterile filter tips for micro pipets
- Sterile microtubes
- Disposable gloves, powderless
- Biohazard waste container
- Refrigerator and freezer
- Tube racks

**7. Warnings and Precaution**

- Carefully read this instruction before starting the procedure.
- For in vitro diagnostic use only.
- This assay needs to be carried out by skilled personnel.
- Clinical samples should be regarded as potentially infectious materials and should be prepared in a laminar flow hood.
- This assay needs to be run according to Good Laboratory Practice.
- Do not use the kit after its expiration date.
- Avoid repeated thawing and freezing of the reagents, this may reduce the sensitivity of the test.
- Once the reagents have been thawed, vortex and centrifuge briefly the tubes before use.
- Prepare quickly the Reaction mix on ice or in the cooling block.
- Set up two separate working areas: 1) Isolation of the RNA/ DNA and 2) Amplification/ detection of amplification products.
- Pipets, vials and other working materials should not circulate among working units.
- Use always sterile pipette tips with filters.
- Wear separate coats and gloves in each area.
- Do not pipette by mouth. Do not eat, drink, smoke in laboratory.
- Avoid aerosols.

**8. Sample Collection, Storage and Transport**

- Collected samples in sterile tubes.
- Specimens can be extracted immediately or frozen at -20°C to -80°C.
- Transportation of clinical specimens must comply with local regulations for the transport of

etiologic agents

**9. Procedure**

**9.1 RNA-Extraction**

RNA extraction kits are available from various manufacturers. You may use your own extraction systems or the commercial kit based on the yield. For the RNA extraction, please comply with the manufacturer's instructions. The recommended extraction kit is as follows:

Nucleic Acid Isolation Kit	Cat. Number	Manufacturer
RNA Isolation Kit	EM-0100/EM-2100	Life Tech
Qiagen RNeasy Mini Kit	74106	QIAGEN

**9.2 Quantitation**


The kit can be used for quantitative or qualitative real-time RT-PCR. A positive control defined as 1x10<sup>7</sup> copies/ml is supplied in the kit.

For performance of quantitative real-time PCR, Standard dilutions must be prepared first as follows.

Molecular Grade Water is used for dilution.

Dilution is not needed for performance of qualitative real-time PCR.

Take positive control (1x10<sup>7</sup> copies/ml) as the starting high standard in the first tube. Respectively pipette 36µl of Molecular Grade Water into next three tubes. Do three dilutions as the following figures:

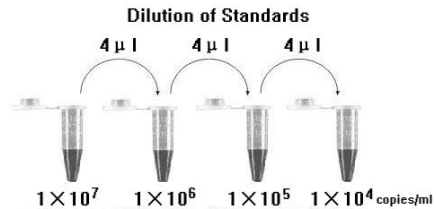


To generate a standard curve on the real-time system, all four dilution standards should be used and defined as standard with specification on the corresponding concentrations.

**Attention:**

- Mix thoroughly before next transfer.
- The positive control (1x10<sup>7</sup> copies/ml) contains high concentration of the target DNA. Therefore, be careful during the dilution in order to avoid contamination.

**9.3 RT-PCR Protocol**

The Master Mix volume for each reaction should be pipetted as follows:

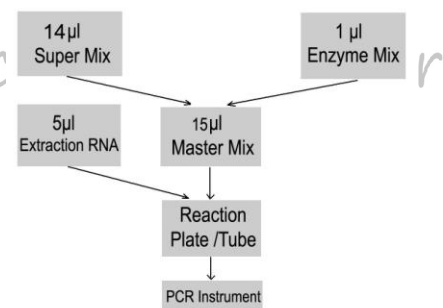

- The volumes of Super Mix and Enzyme Mix per reaction multiply with the number of samples, which includes the number of controls and sample prepared. Molecular Grade Water is used as the negative control. For reasons of unprecise pipetting, always add an extra virtual sample. Mix completely then spin down briefly in a centrifuge. Mix completely then spin down briefly in a centrifuge.
- Pipet 15µl Master Mix with micropipets of sterile filter tips to each of the real time PCR reaction plate/tubes. Separately add 5µl RNA sample, positive and negative controls to different plate/tubes. Immediately close the plate/tubes to avoid contamination.
- Spin down briefly in order to collect the Master Mix in the bottom of the reaction tubes.
- Perform the following protocol in the instrument:

45°C for 20min	1 cycle
95°C for 5min	1 cycle
95°C for 5sec, 58°C for 20 sec, 72°C for 35 sec	45 cycles

(Fluorescence measured at 58°C)

Selection of fluorescence channels	
530nm	Target Nucleic Acid

**10. Threshold setting:** Choose Arithmetic as background and none as Noise Band method, then adjust the Noise band just above the maximum level of molecular grade water, and adjust the threshold just under the minimum of the positive control.

**11. Calibration for quantitative detection:** Input each concentration of standard controls at the end of run, and a standard curve will be automatically formed.

**12. Quality control:**

Negative control, positive control, QS and must be performed correctly, otherwise the sample results is invalid.

Channel	Crossing point value
Control	530nm
Molecular Grade Water	Blank
Positive Control (qualitative as say)	≤35
QS (quantitative detection)	Correlation coefficient of QS curve ≤ -0.98

**13. Data Analysis and Interpretation**

The following results are possible:

	Crossing point value	Result Analysis
1	Blank	Below the detection limit or negative
2	≤43	Positive; the sample contains JAK2-V617F mutant gene; and the software displays the quantitative value
3	43~45	Re-test; If it is still 43~45, report as I#